FR

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 Alliance Pharma (Ireland) Ltd, United Drug House, Magna Drive, Dublin, D24 X0CT, Ireland

Brevet US en instance 13/548,899

*Gold MH et al. Dermatol Surg 2014;40: 825-831

EN

ADVANCED FORMULA SCAR TREATMENT

KELO-COTE® UV is a patented, lightweight, self-drying silicone gel for the treatment of scars. **KELO-COTE® UV** rapidly dries to form a flexible, waterproof sheet which forms a protective barrier against chemical, physical and microbial invasion of the scar site while assisting with hydration. This creates an environment which allows the scar to mature through normalised collagen synthesis and helps improve the physiological and cosmetic appearance of the scar.

In addition, **KELO-COTE® UV** helps protect scars from the aggravating effects of the sun, by preventing sun-induced redness and hyper-pigmentation while improving the overall appearance of the scar. It is recommended that scars are further protected with sunscreen applied every 2 hours during sun exposure. Babies and young children should be kept out of direct sunlight.

KELO-COTE® UV has been shown to flatten, soften and smooth scars, relieve the itching, discomfort and pain of scars, as well as reduce the discolouration associated with scars.

Silicone gels have been shown to be effective in treating abnormal scarring in the form of hypertrophic scars and keloids. The 2014 International Clinical Recommendations on Scar Management supports the use of silicones as the preferred first line therapy for the treatment of linear hypertrophic scars, minor keloids and in the prevention of hypertrophic scars.*

Intended use:

KELO-COTE® UV is for the treatment and prevention of hypertrophic and keloid scars resulting from burns, general surgical procedures and trauma wounds.

KELO-COTE® UV is suitable for use on the face and body. **KELO-COTE® UV** can be used on joints, flexures, hairy areas and large area scars such as those caused by burns or major trauma.

KELO-COTE® UV is suitable for use on children over the age of 3 months.

Directions for use:

Ensure that the affected area is clean and dry. Cut the tip of the nozzle with clean scissors before application. Apply evenly in a thin layer twice daily using a clean finger. **KELO-COTE® UV** should be applied 30 minutes before the first sun exposure each day. Additional sunscreen should be reapplied at least every 2 hours. For maximum effect, **KELO-COTE® UV** should have 24 hour contact with the skin. Once dry, additional sunscreen can be applied on top of the product and **KELO-COTE® UV** can be covered with cosmetics. Replace cap after use.

If **KELO-COTE® UV** has not dried within 4-5 minutes, you have probably used too much. Gently remove the excess and allow the drying process to continue.

Recommended minimum treatment time is 60-90 days. Larger and older scars may take longer and continued use is recommended if improvement is still seen after the initial 90 days.

Caution:

For external use only. Do not use if you are hypersensitive to any of the ingredients in **KELO-COTE® UV**. Avoid direct contact with eyes, mucous membranes, third degree burns and open wounds. **KELO-COTE® UV** should not be used on dermatological conditions that disrupt the integrity of the skin. If irritation occurs, discontinue use and consult your physician.

KELO-COTE® UV may stain clothing if it is not completely dry. Keep away from sources of ignition and do not smoke while applying the product or while the product is drying.

Keep out of the reach of children. Store below 25°C. Do not use after the expiration (EXP) date printed on the tube.

Ingredients: Dimethicone, Cyclopentasiloxane, Octocrylene, Isopropyl Myristate, Ethylhexyl methoxycinnamate, Benzophenone-3, Ethylhexyl salicylate, Silica, Dimethicone Crosspolymer

Contents: 6g, 15g & 30g

Lot number and expiry date are printed on the base of the carton.

www.kelo-cote.co.uk

Manufacturer:

 Advanced Bio-Technologies, Inc, 4830 West Kennedy Blvd, Suite 600, Tampa, Florida 33609, USA

 Alliance Pharma (Ireland) Ltd, United Drug House, Magna Drive, Dublin, D24 X0CT, Ireland

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