

Instructions for use



Anaftin[®] Spray

Anaftin[®] spray helps in the management of pain caused by small lesions in the mouth, such as those due to aphthous stomatitis, aphthous ulcers, as well as small lesions from braces and ill-fitting dentures. It is also indicated for diffuse aphthous ulcers.

Mode of action:

Anaftin[®] Spray contains polyvinylpyrrolidone (PVP) and hyaluronic acid, which form a protective film that adheres to the oral cavity, providing an effective barrier to the affected areas, thus relieving the pain caused by exposed nerve endings. The hyaluronic acid and Aloe vera in the formulation support the natural healing of damaged tissues.

Instructions for use:

When using the product for the first time, take the cap off and apply spray nozzle. Direct the nozzle towards the lesion inside the mouth and spray 3 times or as needed to cover the affected area. For best results, avoid eating or drinking for at least 1 hour after application. The nozzle allows the product to cover the oral lesions to maximum effect.

Use 3 – 4 times a day, or more often if needed.

Ingredients:

Aqua, propylene glycol, polyvinylpyrrolidone (PVP), maltodextrin, *Aloe barbadensis*, potassium sorbate, sodium benzoate, PEG-40 hydrogenated castor oil, disodium EDTA, benzalkonium chloride, sodium hyaluronate, aroma, sodium saccharin, dipotassium glycyrrhizate

Cautions:

The use of Anaftin[®] spray is contra-indicated in any patient with a known history of hypersensitivity to any of the ingredients. If you're pregnant ask a doctor before using the product. If you frequently suffer from mouth ulcers, it recommended that you seek medical advice.

Precautions and warnings:

Do not use after the expiry date, or if the bottle appears to be damaged or is leaking. Keep out of the sight and reach of children. Store at room temperature, away from direct heat. Do not freeze. Avoid contact with the eyes.

Contents:

15 ml

Manufacturer:

Alliance Pharma Srl,
Viale Restelli 5,
20124 Milan,
Italy



Distributed by

BERLIN-CHEMIE AG (MENARINI GROUP),
Glienicker Weg 125,
12489 Berlin,
Germany

Rev 09/2016

engl. / V128770/01
TR1275122 0ALL2FI00048