

opus
stoma care

skinsafe®
medical
protective film
non sting spray



REF 6601 50ml
Expiry date, batch code and UDI
printed on the base of the can



skinsafe® non-sting protective film is intended to form a barrier between the skin around the stoma (colostomy, ileostomy or urostomy) and the adhesives incorporated into flanges and pouches to help prevent skin damage from skin stripping and body fluids.

Instructions for Use: Use on intact skin only. Avoid contact with stoma. Spray on clean dry skin where a stoma pouch or flange is to be applied. Wait for **skinsafe®** to dry and apply stoma pouch or flange as normal.

Contains: Hexamethyldisiloxane, a blend of silicone copolymers and a hydrocarbon propellant.

Warnings and Precautions: For external use only. Keep out of the sight and reach of children. Do not use if sensitive to any of the ingredients. Avoid contact with eyes - if contact occurs flush with water. If skin irritation occurs, cease use and flush with water. Use in a well-ventilated area, seek medical attention in case of respiratory complaints after inhalation. Extremely flammable aerosol. Pressurised container may burst if heated. Keep away from heat, hot surfaces, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition source. Do not pierce or burn even after use. Protect from sunlight, do not expose to temperatures exceeding 50°C/122°F. Dispose of large quantities of unused product responsibly and according to local requirements.

Report any safety concerns to Alliance Pharmaceuticals Limited (tel: +44(0) 1249 466 966, email: pharmacovigilance@alliancepharma.co.uk).

Storage: Store in a cool dry place below 25°C.

