

opus  
stoma care

absorbagel®  
odour minimising sachets



Alliance Pharmaceuticals Limited, Avonbridge House, Bath Road, Chippenham, Wiltshire, SN15 2BB, UK (T/A Opus Healthcare)

EC REP Alliance Pharma (Ireland) Ltd, United Drug House, Magna Drive, Dublin, D24 X0CT Ireland

absorbagel®, Opus, Alliance and associated devices are registered trademarks of Alliance Pharmaceuticals Limited.  
©Alliance Pharmaceuticals Limited 2025



F0030 5 036631 011462 TR3773162

Sachet containing absorbent granules that solidifies stoma output into a gel, reducing leakage, noise and odour, whilst simplifying emptying and draining the stoma pouch. Intended for patients with a temporary or permanent stoma; but not intended for urostomy patients.

**Contains: Sodium Polyacrylate and Polyvinyl Alcohol**

**Instructions for use:** For external use only. Do not open individual sachets. Insert one sachet into ostomy pouch prior to use, ensuring that the sachet is correctly positioned at the bottom of the pouch. When emptying/draining pouches dispose of the contents in the usual manner. Repeat after draining ostomy pouch. Do not use if sachet is damaged or opened. Replace tub lid securely after use. Dispose of package and large quantities of unused product responsibly and according to official regulations.

**Warnings and precautions:** DO NOT EAT. In case of accidental ingestion, drink plenty of water and seek immediate medical attention. In case of accidental eye contact, rinse opened eye with running water and seek medical attention.

Always monitor your stoma output volume closely and seek medical attention about sustained increases. Report any safety issues to:

Alliance Pharmaceuticals Limited  
Tel: +44(0) 1249 466 966  
Email: [pharmacovigilance@alliancepharma.co.uk](mailto:pharmacovigilance@alliancepharma.co.uk)

**Storage:** Store in a cool dry place, below 25°C. Keep out of reach and sight of children.

[www.opus-healthcare.co.uk](http://www.opus-healthcare.co.uk)

opus  
stoma care

absorbagel®  
odour minimising sachets  
for ostomy pouches

150 sachets  
REF 9900

