
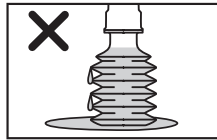
INSTRUCTIONS FOR USE

To minimise the risk of cross infection the contents of each container should be used for only one patient. For single use only. Use immediately after opening. Discard after use. Not for injection.


1. Check that the contents are clear and that the flexible container is intact. Do not use if there is any evidence of leakage or interference with the tamper evident feature of the cap.

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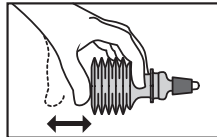5. A controlled flushing action can be achieved by gently compressing and releasing the bellows continuously for **approximately one minute**. Allow the fluid to gently drain back into the container after the agitation period.

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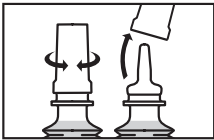2. Twist cap in either direction to break the seal and remove cap.

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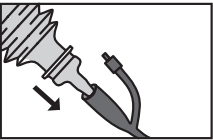
6. Discharge the contents of the container down a toilet, sluice or equivalent and dispose of the container. **NOTE:** Studies* have suggested that the application of 2 solutions used sequentially will dissolve significantly more encrustation than a single application. If 2 solutions are to be administered, repeat steps 1-7.

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
3. Insert the nozzle of the container into the catheter.

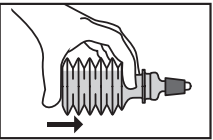3. Breng de tip van de flacon in de katheter.

4. Hold container neck between first and second fingers and squeeze the bottle by pressing on the base with the thumb to provide a controlled flow into the catheter without the introduction of excess air.

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Patient Information Leaflet

ENG

Please read all of this leaflet carefully before using OPTIFLO® G. If you have any questions or are not sure about the use of this irrigation solution then ask your doctor, nurse or pharmacist.

What is OPTIFLO® G?

OPTIFLO® is a range of different irrigating solutions in a specially designed flexible container. OPTIFLO® products are intended to be used by you, your doctor or your nurse in the routine care of your indwelling urinary catheter. Although modern catheters are remarkably free of problems, they still need care if they are to work effectively. The different solutions in the OPTIFLO® range have been designed to deal with the occasional problems which are encountered. All OPTIFLO® products are supplied sterile.

OPTIFLO® G contains a solution called Suby G. This solution can help keep your catheter free of deposits of crystals which might otherwise lead to blockage and infections in your bladder. Suby G can also help in the removal of crystal deposits which have already built up in your catheter. Water alone is not effective in removing these deposits.

OPTIFLO® G is provided in two sizes – 50ml and 100ml. Each bottle contains Suby G, which consists of citric acid (3.23% w/v), light magnesium oxide (0.38% w/v), sodium bicarbonate (0.7% w/v) and disodium edetate (0.01% w/v) in water for injections Ph.Eur.

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Contents: **50ml, 100ml**
Inhoud: 50 ml, 100 ml

This is a medical device.

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Becton Dickinson Distribution Center NV, Laagstraat 57, 9140 Temse, Belgium
Becton Dickinson Distribution Center Forest House, Tilgate Forest Business park, Brighton Road, Crawley, West Sussex, RH11 9BP, UK

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Alliance Pharma S.r.l.
Viale Restelli 5,
20124 Milano, Italy

How should I use OPTIFLO® G?

Follow the instructions given by your doctor or nurse. You will find a step-by-step guide printed on the other side of this leaflet. This will show you how OPTIFLO® G should be used. For comfort, OPTIFLO® G should be at room temperature when instilled.

Hygiene is very important in the care of indwelling urinary catheters. Remember to wash your hands thoroughly with soap and water before and after handling your catheter or any of the parts of the collection bag and drainage tube.

Your doctor or nurse will work out how often you should use OPTIFLO® G. Your catheter care plan will be tailored to your needs as an individual. Don't worry if you know someone else with a catheter who has a different care plan. You should never use more than your doctor or nurse recommends.

Are there any special precautions you should take when using OPTIFLO® G?

Only use OPTIFLO® G as your doctor or nurse recommends.

Do not share OPTIFLO® G with anyone else.

Do not use OPTIFLO® G after the expiry date shown on the outside of the carton and on the underside of the container.

Do not use OPTIFLO® G if the cap has been disturbed, or if the container shows signs of leakage. If the container is punctured, return the pack and contents to your pharmacy.

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Are there any side effects?

OPTIFLO® G may cause some unwanted effects. Rarely, OPTIFLO® G can cause a slight burning sensation. If this occurs, tell your doctor or nurse.

What to do if OPTIFLO® G is accidentally swallowed

Contact your doctor, nurse or pharmacist immediately.

How should OPTIFLO® G be stored?

Not for injection. Do not use if pack is damaged. Keep out of reach and sight of children. Store in carton, where supplied, below 25°C, in a clean dry place. Use immediately after opening.

Further information

OPTIFLO® G is for single use. Always dispose of OPTIFLO® G carefully after use.

If you experience any other difficulty or problem when using OPTIFLO® G, contact your doctor or nurse.

Although all OPTIFLO® products are intended for use in catheter care, other products in the range have different applications. If your doctor or nurse has recommended that you should use more than one product from the OPTIFLO® range, ensure that you are using the correct solution at all times.

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OptiFlo is a registered trademark of Becton, Dickinson and Company.