Anaftin[®] Baby Teething Gel

For teething pain relief

Intended use:

Anaftin[®] Baby Teething Gel is intended to be used in the oral cavity for relieving the discomfort and pain associated with teething: eruption of a tooth. Anaftin[®] Baby Teething Gel rapidly forms an invisible film which helps to protect gums from the pain and discomfort associated with teething. Its ingredients help to reduce pain and redness in affected areas. During use, the applicator, equipped with silicone bristles, gently massages the sore, irritated gums and may provide temporary pain relief during application. This makes the application of the gel less distressing for the infant and easier for the parent to apply the product. Anaftin[®] Baby Teething Gel has a pleasant banana flavour. Anaftin[®] Baby Teething Gel is lactose, sugar, alcohol and local anaesthetic free.

Instructions for use:

Remove cap, squeeze a pea sized amount and keep the tube compressed. Apply to the affected area and gently massage gums with the applicator. When reapplying, before using the product, remove any dried residual from the applicator with clean dry hands. In infants, apply preferably after meals and before baby's sleep to promote film build up and prolong pain relief. Use 3-4 times a day as needed. Return cap to the top of the applicator after each application. Anaftin[®] Baby Teething Gel is suitable for use in children over the age of 3 months.

All babies teethe at different rates, but this is a general guide of what to expect.

Ingredients:

Aqua, PVP (12%), maltodextrin, propylene glycol, PEG-40 hydrogenated castor oil, aloe barbadensis leaf extract, crocus sativus extract, xanthan gum, aroma, potassium sorbate, sodium benzoate, sodium hyaluronate (0.1%), xylitol, disodium EDTA, cetylpyridinium chloride, glycyrrhetic acid.

Contraindications, Warnings, and Precautions of use:

Each Anaftin[®] Baby Teething Gel tube should be used in a single patient to avoid cross contamination of the applicator and gel. The use of Anaftin[®] Baby Teething Gel is contraindicated for patients with a history of hypersensitivity to any of the ingredients. Do not use if you're pregnant or think you're pregnant or lactating. Keep out of reach of children. **The product should only be applied by an adult. CAUTION:** This medical device is not a toy. **NEVER** leave it in the hands of a child to avoid risk of choking. Check the medical device before use for any signs of wear and tear and pay attention during the application of the gel that there is no accidental detachment of any of the component parts of the applicator head. In case discontinue use. Do not try to reassemble the detached part. Do not use if the tube is open or damaged. Any incident experienced whilst using this product should be reported to the manufacturer pharmacovigilance@alliancepharma.co.uk. Serious deterioration of health that has occurred in relation to this product should also be reported to the competent authority of your country.

Contents: 10ml e tube.

Store between 5°C and 25°C. Stop using the device once gel has run out. Do not use after expiry date.

Rev 1: 2024-08

 Alliance Pharmaceuticals Ltd.
Avonbridge House, Bath Road,
Chippenham, Wiltshire SN15 2BB,
United Kingdom

 Alliance Pharma (Ireland) Ltd.
United Drug House, Magna Drive,
Dublin D24 XOCT, Ireland

 Distributed by: Berlin-Chemie AG
Glienicke Weg 125
12489 Berlin, Germany

    Medical device
www.alliancepharmaceuticals.com

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